


Intended Use:

Guardian cases are made of soft silicone intended to protect specific transmitter patient monitoring devices.

Installation::

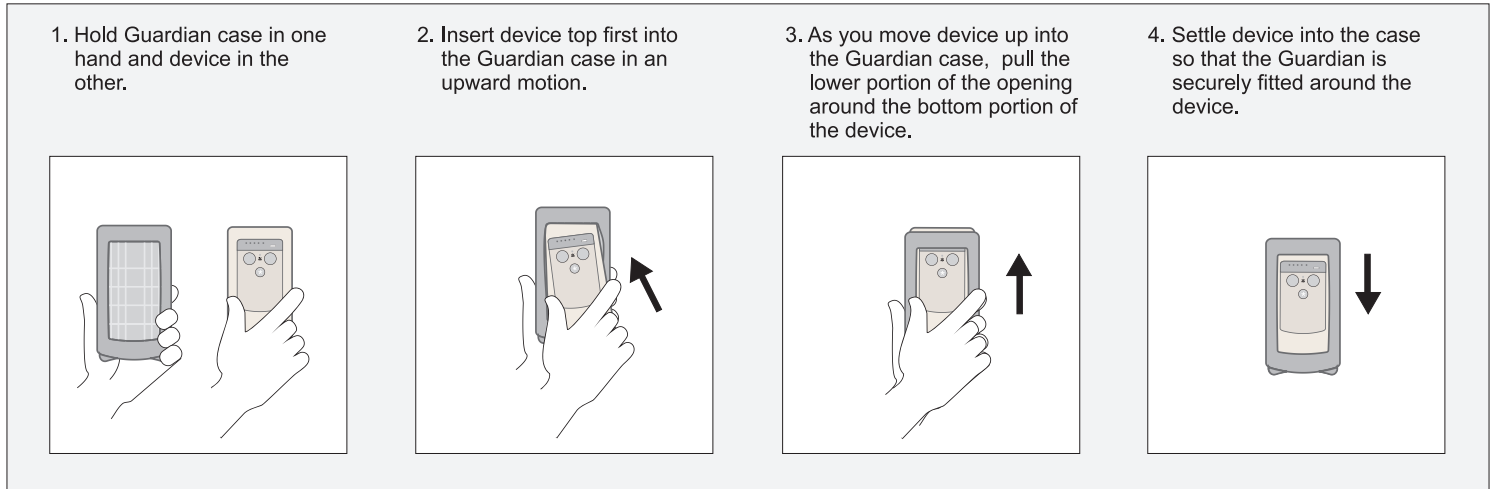

Cleaning Recommendations:

Your Guardian should be cleaned on a regular basis or if it becomes heavily soiled during patient use. Before cleaning the Guardian, consult your hospital's guidelines for cleaning and disinfecting. Use only the substances approved and methods listed to clean or disinfect your Guardian. Warranty does not cover damage caused by unapproved substances or methods. We make no claims regarding the efficacy of the listed chemicals or methods as a means for controlling infection. For methods to control infection, consult your hospital's Infection Control Officer or Epidemiologist.

The Guardian can be cleaned by two methods: wiping or soaking.

WIPING Method of Cleaning:	
STEP 1	Remove the transmitter and any cables or accessories from the Guardian.
STEP 2	Wipe all surfaces of the Guardian clean by using a cloth dampened modestly with one of the following approved cleaning agents: <ul style="list-style-type: none"> • Mild soap and Water • Isopropyl Alcohol (≥ 70%) • Hydrogen Peroxide, 3% solution • Diluted Chlorine Bleach (Sodium hypochlorite), 10% solution prepared within 24 hours • Lysol® • Windex® • Cidex® • CaviCide™ • PDI® Super Sani-Cloth® Wipes
STEP 3	Wipe all cleaned surfaces with distilled water to remove any cleaning solution/residue.
STEP 4	Allow to air-dry, or dry with a lint-free cloth.

SOAKING Method of Cleaning:	
STEP 1	Remove the transmitter and any cables or accessories from the Guardian.
STEP 2	Soak the Guardian in one of the following approved cleaning agents for up to five minutes: <ul style="list-style-type: none"> • Mild soap and Water • Isopropyl Alcohol (≥ 70%) • Hydrogen Peroxide, 3% solution • Diluted Chlorine Bleach (Sodium hypochlorite), 10% solution prepared within 24 hours • Lysol® • Windex® • Cidex® • CaviCide™
STEP 3	Wipe all cleaned surfaces with distilled water to remove any cleaning solution/residue.
STEP 4	Allow to air-dry, or dry with a lint-free cloth.

guardian Case Styles

- Custom fit to each device for ensured protection
- Simple to clean and disinfect
- Shock absorbing internal frame for durability
- Soft shell construction for patient comfort
- Single-piece construction: no lost components
- Made of recyclable material
- Lint free/dust free coating
- Protects your investment
- Easy installation



PHILIPS

Case

Product #	Transmitter Model
CPH01.....	M4841A (S01-ECG Only)
CPH02.....	M4841A (S02-ECG & SpO2)
CPH02.....	M4841A (S03-ECG & SpO2 Upgradeable)
CPH01.....	M2601B (S01-ECG Only)
CPH02.....	M2601B (S01-ECG & SpO2)
CPH02.....	M2601B (S03-ECG & SpO2 Upgradeable)
CPH03.....	MX40
CPH04.....	M2601A



SPACELABS

Case

Product #	Transmitter Model
CSL01.....	90341-05 (ECG Only)
CSL02.....	90343-05 (Multiparameter)
CSL01.....	90347-05 (ECG Only)
CSL03.....	91341-09 (ECG Only)
CSL04.....	91343-09 (Multiparameter)
CSL03.....	91347-09 (ECG Only)



Case

Product #	Transmitter Model
CDM01.....	Telepack 608
CDM02.....	DT-4500



GE|MARQUETTE

Case

Product #	Transmitter Model
CGE01.....	Apex Pro
CGE01.....	Apex Pro CH
CGE02.....	Apex Pro FH

Warranty:

Sage Services Group warrants that this product (the 'Guardian') will be free from defects in workmanship and materials under normal use for a period of one year from the original purchase date. This limited one-year warranty does not cover misuse, lack of care, damage caused by improper maintenance or product that has been modified or altered. This warranty does not cover, under any circumstance, the replacement or repair cost of any medical device inside or outside of a protective case manufactured by Sage Services Group.



506 Deanna Lane
 Charleston, SC 29492
www.sageservicesgroup.com
 Toll Free: 877-281-SAGE (7243)
 Office: 843-284-5844
 Fax: 866-345-5544

Rev. 2015.12.14
 Made in China